


OMNIA Touchscreen Control Guide

REF/KRT MODE

Measurement Screen

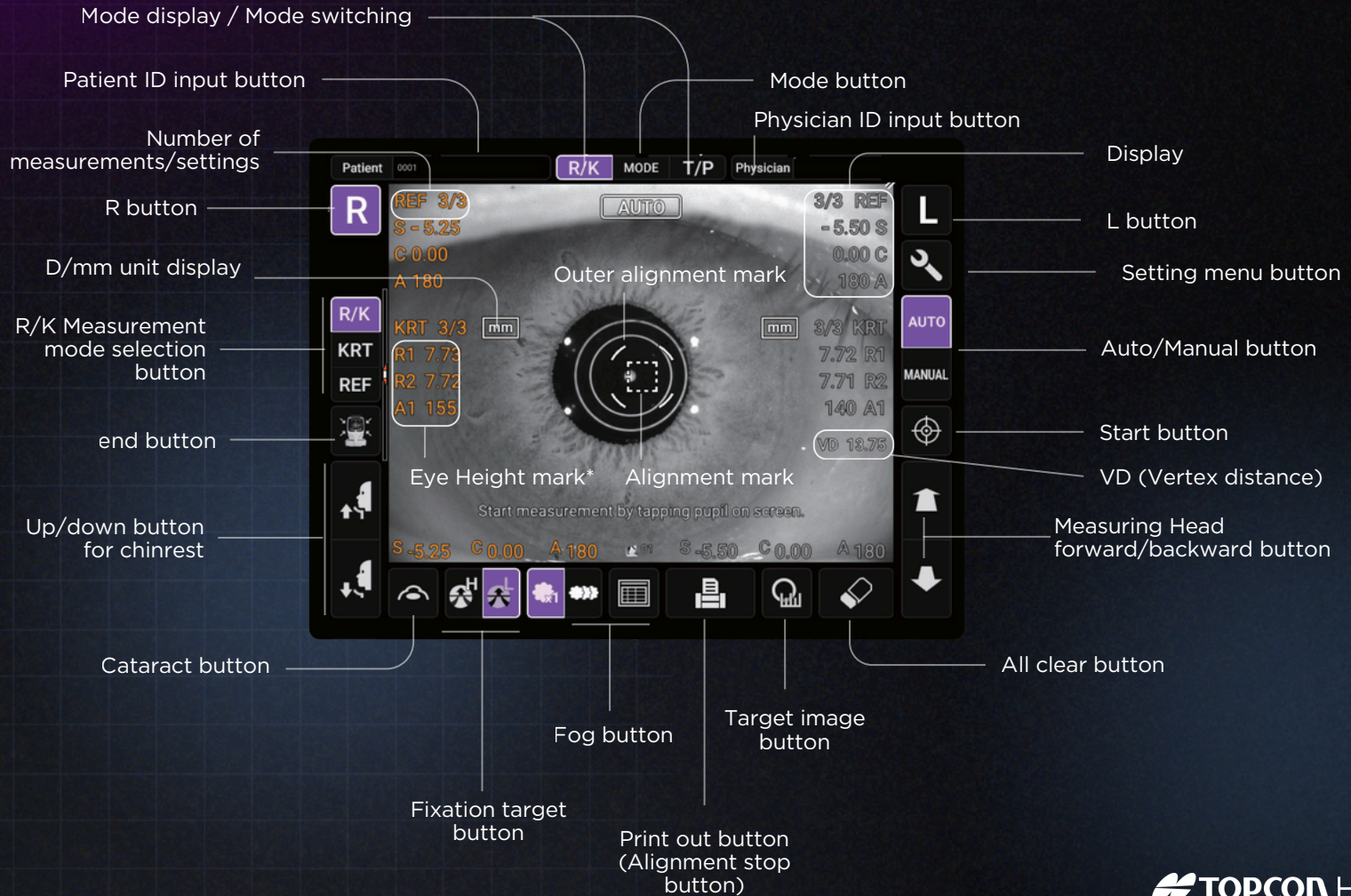

* Eye Height mark: Shows the position of the eye height mark on the chinrest.



MODE DISPLAY / MODE SWITCHING

Switch the mode to R/K measurement or T/P measurement.



PATIENT ID INPUT BUTTON

Displays the software keyboard. Enter the ID name, etc.




PHYSICIAN ID INPUT BUTTON

Displays the software keyboard. Enter the ID name, etc. Displayed when "Physician ID" and "Use Physician ID" are set to ON on the Setup screen.



R/L BUTTON

Selects the right/left eye. By tapping the button, the main body moves to the selected direction. The selected button is framed in purple. The layout where the R/L button is displayed reverses according to the position of the control panel.

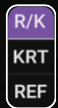

R/K MEASUREMENT MODE SELECTION BUTTON

Selects each measurement REF, KRT.



UP/DOWN BUTTON FOR CHINREST

Moves the chinrest up/down.



END BUTTON

The chinrest and measuring head move to the last position. Turn off the power when the movement to the last position has completed.



MEASURING HEAD FORWARD/BACKWARD BUTTON

Moves the measuring head closer to/away from the patient's eye. The forward and backward operations reverse according to the position of the control panel.



START BUTTON

Starts measurement.



AUTO/MANUAL BUTTON

Switches between Auto mode and Manual mode. If "AUTO" is displayed on the control panel it is in Auto mode, if "MANUAL" is displayed it is in Manual mode.



MODE BUTTON

Selects and sets each measurement of REF, KRT, TONO, and PACHY.



CATARACT BUTTON

If it is difficult to measure due to cataract (lens opacity) or obstruction of eyelids, eyelashes, blinking, etc., tap this button to enable cataract mode and adjust the amount of light to make measurement easier. The CAT icon appears in the upper part of the control panel, and the button frame changes to purple.



FIXATION TARGET BUTTON

Brightness of the fixation target can be changed.



FOG BUTTON

Changes setting temporarily to perform fogging only in the first measurement or each time in the continuous measurement.



ALL DATA BUTTON

Displays all measurement data on the screen.



PRINT OUT BUTTON

Prints and/or exports measurement results. Tap the button when no measurement data is present to feed the paper.



ALIGNMENT STOP BUTTON

Displayed during an alignment or measurement, the alignment or measurement operation is stopped.



TARGET IMAGE BUTTON

Opens the target image view screen.



ALL CLEAR BUTTON

Clears all measurement data.




SETTING MENU BUTTON

Shifts to the setting menu screen.

OMNIA Touchscreen Control Guide

TONO / PACHY MODE

Measurement Screen

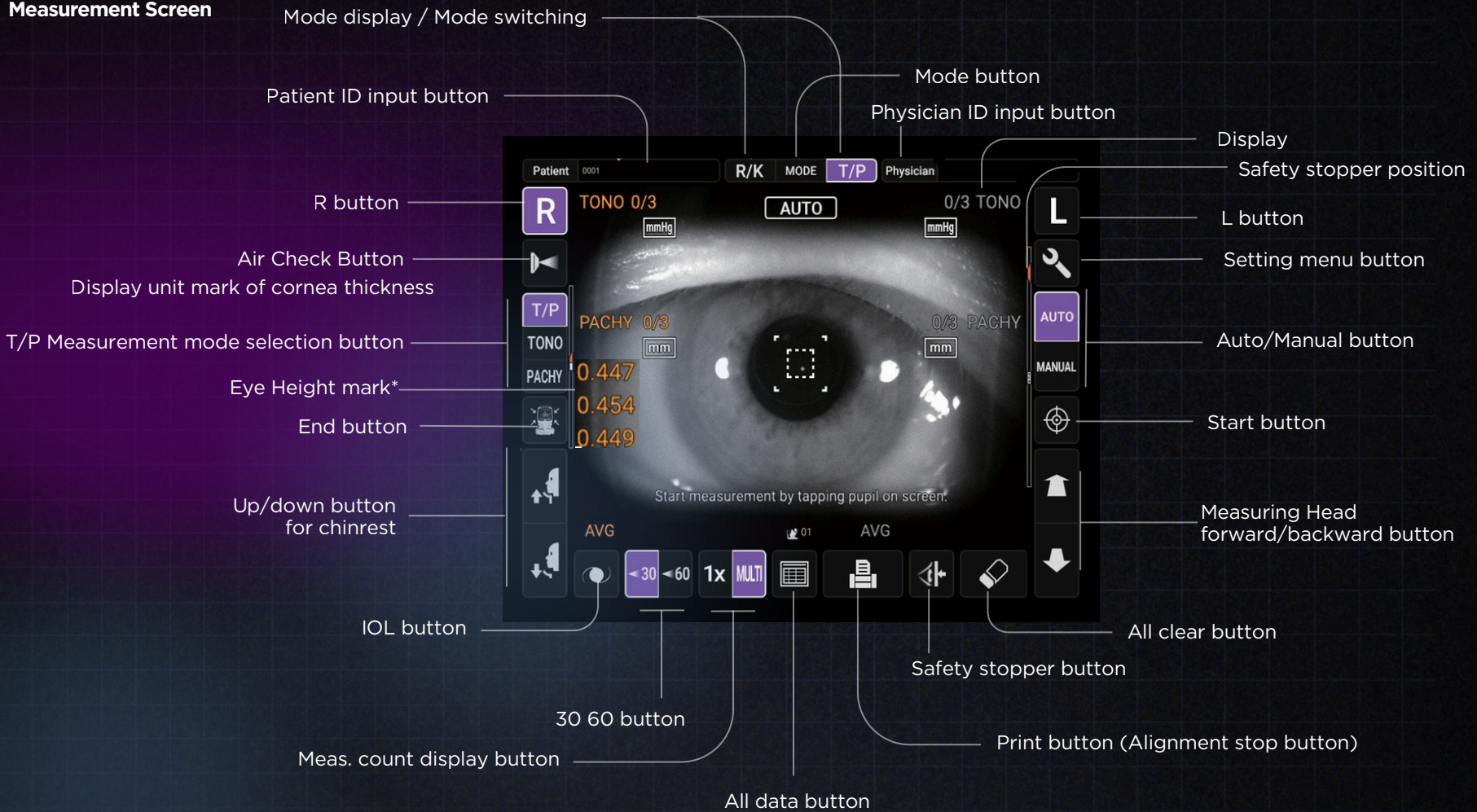

* Eye Height mark: Shows the position of the eye height mark on the chinrest.



MODE DISPLAY / MODE SWITCHING

Switch the mode to R/K measurement or T/P measurement.



PATIENT ID INPUT BUTTON

Displays the software keyboard. Enter the ID name, etc.




PHYSICIAN ID INPUT BUTTON

Displays the software keyboard. Enter the ID name, etc. Displayed when "Physician ID" and "Use Physician ID" are set to ON on the Setup screen.




R/L BUTTON

Selects the right/left eye. By tapping the button, the main body moves to the selected direction. The selected button is framed in purple. The layout where the R/L button is displayed reverses according to the position of the control panel.

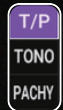

R/K MEASUREMENT MODE SELECTION BUTTON

Selects each measurement TONO, PACHY.

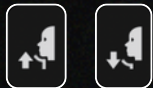

T/P BUTTON FOR CHINREST

Moves the chinrest up/down.



AIR CHECK BUTTON

Perform air check. Check if the measurement system inside the instrument is working properly.



MEASURING HEAD FORWARD/BACKWARD BUTTON

Moves the measuring head closer to/away from the patient's eye. The forward and backward operations reverse according to the position of the control panel.



START BUTTON

Starts measurement.



AUTO/MANUAL BUTTON

Switches between Auto mode and Manual mode. If "AUTO" is displayed on the control panel it is in Auto mode, if "MANUAL" is displayed it is in Manual mode.




MODE BUTTON

Selects and sets each measurement of REF, KRT, TONO, and PACHY.




IOL BUTTON

If alignment does not work with a patient who has an IOL, tapping this button may allow the measurement. When this button is selected, the IOL icon appears in the upper part of the control panel, and the button frame changes to purple. Sets the brightness of the LED for IOL.

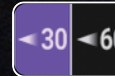

30/60 BUTTON

Switches the range of intraocular pressure values to be measured between 7-30 mmHg and 30-60 mmHg.

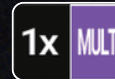

MEAS. COUNT CHANGE BUTTON

Switches the measurement count between one-time measurement and preset measurement count.
- Preset measurement count (Runs the preset number of measurements)
- One-time measurement (This is the factory default setting)



ALL DATA BUTTON

Displays all measurement data on the screen.



PRINT OUT BUTTON

Displayed when the instrument is in measurement standby. Use this button to print out measurement results or output data to the network. Tap the button to feed the paper if there are no measured values.



ALIGNMENT STOP BUTTON

Displayed during an alignment or measurement, the alignment or measurement operation is stopped.



ALL CLEAR BUTTON

Clears all measurement data.



SAFETY STOPPER BUTTON

Switches to the setting screen of the safety stopper position to prevent the measuring window glass from hitting the patient's eye during the measurement.



SETTING MENU BUTTON

Shifts to the setting menu screen.